

INSTRUCTIONS FOR USE

Biodent TS

HERBAL TOOTH VARNISH

**TEMPORARY DENTAL SEALANT
BASED ON HERBAL RESIN**



Use Biodent TS if you have an increased risk of caries: with frequent sugar use and if you still had a cavity in the past two years. Use Biodent TS to reduce the risk of gingivitis.

Apply once a week, for a maximum of 4 weeks.

Biodent BV

Mollebite 19 • 6931 KE Westervoort • Nederland

www.biodent.nl • info@biodent.nl

PREPARE

- Unscrew the cap from the syringe
- Screw the supplied tip onto the syringe



Apply to dental necks



Apply to chewing surfaces



Apply between teeth, floss

APPLY

- Apply to the tooth surface free from dental plaque and food residues.
- Apply where plaque is difficult to remove with tooth brushing.



Do not eat and drink for 5 minutes and then brush your teeth, spit out small pieces. The visible varnish that cannot be brushed off dissolves in a few hours. Brush gently, don't damage the gums.

CLOSE

After applying, turn the tip of the syringe and close the syringe with the closing cap. An invisible amount of varnish remains in the pores of the tooth surface, then slowly dissolves and thus has an antibacterial effect.